


Studex System 75 Ear Piercing Instructions



1. Peel off the backing paper from the blister. Do Not Remove the Cartridge From The Blister!




2. Holding the sides firmly, slide the cartridge down into the channel in the instrument until it is seated fully.

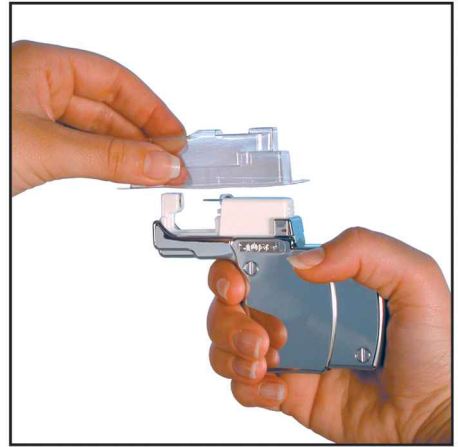


3. Lift off the plastic blister. Do not discard the blister, as you will need it later.



4. Mark the ear (ears) accordingly with nontoxic Studex Marking Pen.

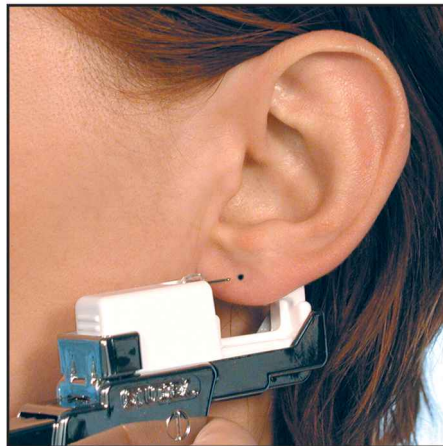

5. Place the ear between the end of the stud and the clasp, aligning the point of the stud with mark on the ear. Slowly squeeze the handle.

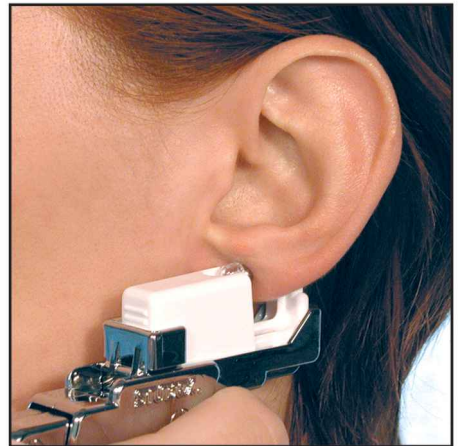


6. Keeping the proper alignment, squeeze the handle firmly to pierce the ear. The clasp will be automatically installed with the proper spacing.



7. Release pressure on the handle and carefully pull the handle straight away from the ear. This will release the stud from the clear plastic carrier and the clasp from the cartridge.



8. After piercing, place the plastic blister (the one that cartridge came in) down on top of the used cartridge as shown.

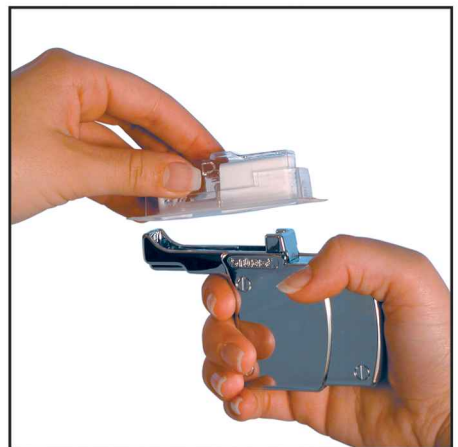

9. Lift the entire cartridge straight up, holding the sides of the blister and cartridge firmly and slightly forward to release the cartridge from the channel. Dispose of the used cartridge and blister in the appropriate manner.

521 Rosecrans Avenue, Gardena, CA 90248-1514, U.S.A.
310.851.9300; 1.800.478.8339; Fax:310.851.9400
www.studex.com • e-mail: sales@studex.com